



PRODUCT SHEET

Bariatric Shower Chair Clean

Strong and functional bedside commode/shower chair intended for bariatric users.

Clean is used in care homes, institutions, and hospitals.
User weight up to 325 kg. Available in two sizes, 610 mm, and 710 mm width.



- More setting options
- Available in two sizes, 610 mm, and 710 mm width
- Weight capacity up to 325 kg
- High-tech lightweight tubes ensuring maximum stability and low weight.



General information

This product sheet contains important information about the use of Shower Chair Clean and the target group for the product. It also includes a range of specifications on Shower Chair Clean.

The most recent version of this product sheet can be found on our website where it also can be scaled up to a larger font size.

You can always contact Cobi Rehab if you want more information about Shower Chair Clean.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Bariatric Shower Chair Clean

Shower Chair Clean is intended for users weighing up to 325 kg during lavatory use and in shower situations.

Shower Chair Clean is used in hospitals, care centers and institutions.

Shower Chair Clean is available in two seat width, 610 mm, and 710 mm.

The seat depth can also be adjusted in three different depths.

Shower Chair Clean may only be used on a flat, hard floor with a maximum inclination of 3°. Larger slope increases the risk of tipping.

Environmental Restrictions

Shower Chair Clean must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.

Contraindications

Monitor users with cognitive impairment while using Shower Chair Clean.

Construction

Shower Chair Clean consists of lightweight tubes ensuring maximum stability and low weight. The pipes are surface treated.

Shower Chair Clean is designed with armrests bending out like a butterfly. Butterfly armrest ensures the user's body does not get squeezed, getting in and out of the shower chair.

Settings

Shower Chair Clean consists of two parts, the chair frame, and the backrest. Shower Chair Clean is assembled using the supplied bolts and washers.


Adjustment of the seat depth and backrest

The seat depth can be adjusted in three positions. Select two holes (front, middle or rear) on the backrest for the desired seat depth.

Cleaning, maintenance, and long-term storage

The frame, seat, and armrests of the Shower Chair Clean must be washed with lime- and salt-neutral water using a regular cleaning agent after each use, and then wiped dry.

Environmental Restrictions

 The Shower Chair Clean must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.

Disinfection

The seat can be removed and disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

The backrest of Shower Chair Clean can be adjusted to achieve the best possible sitting position. The backrest can be moved +/- 60 mm in total.


The seat height is adjustable from 440-590 mm.

Adjustment of the height settings of the legs

The height of the legs can be adjusted in six positions by loosening and tightening the four bolts already mounted on the legs.

During use

The backrest can be adjusted in three different depths. Adjusting the seat depth does not affect the stability of the chair.

 Always remember to disinfect Shower Chair Clean when transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

Long-term storage

When Shower Chair Clean is not in use, store it in a dry place and between -10 and 40°C.

Specifications

Specifications, model 610 mm og 710 mm	
Maximum user weight	325 kg
Seat width	610/710 mm
Seat depth adjustable	400-520 mm
Seat height adjustable	440-590 mm
Distance between arm rests	650/750 mm
Total width	730/830 mm
Total height	790-940 mm
Width toilet opening	Ø 240 mm, 350 mm long
Seat/back/arm rests	Polyurethane
Frame	Powder-lacquered chromium molybdenum steel/SAE41-30
Weight (610 mm/710 mm)	14/15 kg
ISO classification	09 33 07 03
Expected product service life	7 years
Risk class	I
UDI-DI	5740000101288/5740000101295
Basic UDI	5740000100003MY
General requirements and test methods	EN 12182??
The surface temperature can rise when the Shower Chair Clean is exposed to external heat sources e.g., sunlight.	

Ordering

Model	Item number
Bariatric Shower Chair Clean, 610 mm	01420-061-000
Bariatric Shower Chair Clean, 710 mm	01420-071-000
Seat for Shower Chair Clean, 610 mm	0150-999-022
Seat for Shower Chair Clean, 710 mm	0150-999-023
Bedpan with cover	0140-999-001
Ferrules, rubber, 1 piece	0140-999-112