



PRODUCT SHEET

Bariatric Shower Commode Wheeler

Strong shower commode with rear wheels, which contributes to greater independence for the user. Adjustable footrests so users with a lot of tissue on the inside of the thighs can sit with legs abducted.

Shower Commode Wheeler is used in care homes, institutions, and hospitals. Available in two widths, 610 mm, and 710 mm. User weight up to 325 kg.



- Brakes on two wheels. Possibility of brakes on all four wheels.
- Optional extra-wide butterfly arm rests.
- Available in two widths, 610 mm, and 710 mm.
- Shower Commode also ensures a better working environment for the caregiver
- Extra-long arm rests.
- Several types of leg rests.



General information

This product sheet contains important information about the use of Shower Commode Wheeler and the target group for the product. It also includes a range of specifications on Shower Commode Wheeler.

The most recent version of this product sheet can be found on our website where it also can be scaled up to a larger font size.

Bariatric Shower Commode Wheeler

Shower Commode Wheeler is designed to help the bariatric user during transfers to and from the bathroom, in the shower, and during lavatory use.

Shower Commode Wheeler ensures greater independence for the user. Shower Commode Wheeler is intended for indoor use in e.g., hospitals, care centers and institutions.

Shower Commode Wheeler is developed for users up to 325 kg with reduced physical capacity.

Shower Commode Wheeler must only be used on a firm surface to avoid the risk of tipping.

Construction

Shower Commode Wheeler consists of lightweight tubes ensuring maximum stability and low weight. The pipes are surface treated.

Shower Commode Wheeler has two different types of armrests. Standard armrests and armrests bending out like a butterfly, so the user's body does not get squeezed, getting in and out of the shower Commode Wheeler.

Each footrest can carry up to 100 kg and is side and width adjustable making access to intimate hygiene possible.

You can always contact Cobi Rehab if you want more information about Shower Commode Wheeler.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Shower Commode Wheeler can be used in environments with a temperature range of 0-40°C and a humidity interval of 20-80%.

Contraindications

Monitor users with cognitive impairment while using Shower Commode Wheeler.

Environmental Restrictions

The Shower Commode Wheeler must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.

Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

Shower Commode Wheeler has a split toilet seat, measuring 400 mm and 240 mm at the widest point.

The space between backrest and seat leaves space for the gluteal region and enables the caregivers to reach for the sling.

Before use and during use

To prevent the risk of the user falling off the side of Shower Commode Wheeler, never use it without properly installing the armrests first.

During use

Installing the armrests while the user is seated on Shower Commode Wheeler may present a pinch

hazard between the armrests and the side of Shower Commode Wheeler.


Both wheels must be locked with the wheel locks when the user is seated on Shower Commode Wheeler, except when it is used for transport.

Cleaning, maintenance, and long-term storage

The frame, seat, and armrests of the Shower Commode Wheeler must be washed with lime- and salt-neutral water using a regular cleaning agent after each use, and then wiped dry.


Shower Commode Wheeler is washable in a cabinet washer.

Environmental Restrictions

 The Shower Commode Wheeler must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.


Disinfection

The seat can be removed and disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

 Always remember to disinfect Shower Commode Wheeler when transferred to a new

user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

 Failure to inspect bolts and lock nuts may result in malfunction.

Long-term storage

When Shower Commode Wheeler is not in use, store it in a dry place and between -10 and 40°C.

Specifications

Specifications 610 mm/ 710 mm	
Maximum user weight	325 kg
Risk class	1
Basic UDI DI	5740000100004N2
Expected product service life	7 years
Total length	1290 mm
Total width with standard arm rests (without footrests)	810/910 mm
Total width with butterfly arm rests (without footrests)	870/970 mm
Seat width with standard arm rests	610/710 mm
Seat width with butterfly arm rests	750/850 mm
Seat depth	560 mm
Seat height	550 mm
Width toilet opening	Ø 240 mm, 400 mm long
Distance from ground to under the seat	445 mm
Distance from ground to push bar	1000 mm
Distance from seat to the upper edge of the back	450 mm
Distance from seat to footrests	400/420/440/460/480 mm
Total weight (model 610 mm/710 mm)	33 kg
Swing away arm rests	Yes
Footrests swing away, height adjustable	Yes
Footrest material	PU foam
Individual wheel locks	Yes, 4 pieces
Turning radius	1580 mm
Frame	Powder lacquered chromium molybdenum steel
Wheels/Rear wheels	Ø125 mm, massive, 2 pieces/24" massive, 2 pieces
Seat/back/arm rests	Polyurethane
IP classification	IPX4
ISO classification / Risk assessment	09 12 03 24 / EN/ISO 14971:2012
General requirements and test methods	EN 12182
The surface temperature can rise when the Shower Commode Wheeler is exposed to external heat sources e.g., sunlight.	

Ordering

Model	Item number
Shower Commode Wheeler incl. standard armrests, standard footrests, and commode bucket, 610 mm	0152-061-000
Shower Commode Wheeler incl. standard armrests, standard footrests, and commode bucket, 710 mm	0152-071-000
Shower Commode Wheeler incl. butterfly armrests, standard footrests, and commode bucket, 610 mm	0153-061-000
Shower Commode Wheeler incl. butterfly armrests, standard footrests, and commode bucket	0153-071-000
Butterfly arm rest right (adds 7 cm width between the arm rest)	0150-999-012R
Butterfly arm rest left (adds 7 cm width between the arm rest)	0150-999-012L
Commode bucket	0140-999-001
Full size seat for Shower Commode with bucket and cover, 610 mm	0150-999-022
Full size seat for Shower Commode with bucket and cover, 710 mm	0150-999-023
Footrest standard shower commode right	0156-999-090H
Footrest standard shower commode left	0156-999-090V
Calf support/amputation support, right	0156-999-030
Calf support/amputation support, left	0156-999-031