



Product sheet

Bariatric Shower Commode

Strong and wide Shower Commode with space under the backrest for the tissue in the gluteal region. Adjustable footrests so users with a lot of tissue on the inside of the thighs can sit with legs abducted.

Shower Commode is used in care homes, institutions, and hospitals.
User weight up to 325 kg.



- Brakes on two out of four wheels.
- Optional extra-wide butterfly arm rests.
- Available in two sizes, 610 mm, and 710 mm.
- The Shower Commode also ensures a better working environment for the caregiver.
- Extra-long arm rests.
- Several types of leg rests.



General information

This product sheet contains important information about the use of the Shower Commode and the target group for the product. It also includes a range of specifications on Shower Commode.

The most recent version of this product sheet can be found on our website where it also can be scaled up to a larger font size.

Bariatric Shower Commode

The Shower Commode is designed to help the bariatric user (up to 325 kg) during transfers to and from the bathroom, in the shower, and during lavatory use in e.g., hospitals, care centers and institutions.

The Shower Commode must only be used on a firm surface to avoid the risk of tipping.

The Shower Commode can be used in environments with a temperature range of 0-40°C and a humidity interval of 20-80%.

Construction

The Shower Commode consists of lightweight tubes ensuring maximum stability and low weight. The pipes are surface treated.

The Shower Commode has two different types of armrests. Standard armrests and armrests bending out like a butterfly, so the user's body does not get squeezed, getting in and out of the Shower Commode.

Each footrest can carry 100 kg and is side and width adjustable making access to intimate hygiene possible.

You can always contact Cobi Rehab if you want more information about the Shower Commode.

Cobi Rehab reserves the right to change the product specifications without providing prior notice.

Contraindications

Monitor users with cognitive impairment while using the Shower Commode.

Environmental Restrictions

The Shower Commode Wheeler must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.

Never stand on the foot plates or use them as a platform during transfers, as this will pose a high risk of tipping.

The Shower Commode has a split toilet seat, measuring 44 cm and 22 cm at the widest point.

The space between backrest and seat leaves space for the bulbous gluteal region and enables the caregivers to reach for the sling.

Before use and during use

To prevent the risk of the user falling off the side of the Shower Commode, never use it without properly installing the armrests first.

During use

Installing the armrests while the user is seated on the Shower Commode may present a pinch

hazard between the armrests and the side of the commode.


Both wheels must be locked with the wheel locks when the user is seated on the Shower Commode, except when it is used for transport.

Cleaning, maintenance, and long-term storage

The frame, seat, and armrests of the Shower Commode Wheeler must be washed with lime- and salt-neutral water using a regular cleaning agent after each use, and then wiped dry.


Shower Commode Wheeler is washable in a cabinet washer.

Environmental Restrictions


 The Shower Commode Wheeler must not be used in saline environments (such as swimming pools) or in highly acidic or alkaline environments.

Disinfection

The seat can be removed and disinfected using ethanol 70-85% or a chlorine product of min. 1000 ppm and max. 10000 ppm.

 Always remember to disinfect Shower Commode Wheeler when transferred to a new user. This is done to avoid cross-contamination and infections.

Local cleaning and disinfection procedures must always be followed.

 Failure to inspect bolts and lock nuts may result in malfunction.

Long-term storage

When Shower Commode Wheeler is not in use, store it in a dry place and between -10 and 40°C.

Specifications

Specifications 610 mm/ 710 mm	
Maximum user weight	325 kg
Risk class	1
Basic UDI DI	5740000100004N2
Expected product service life	7 years
Total length	1170 mm
Total width with standard arm rests (without footrests)	690/790 mm
Total width with butterfly arm rests (without footrests)	870/970 mm
Seat width with standard arm rests	610/710 mm
Seat width with butterfly arm rests	750/850 mm
Seat depth	560 mm
Seat height	550 mm
Width between arm rests	60 mm
Width toilet opening	Ø 240 mm, 350 mm long
Distance from ground to under the seat	445 mm
Distance from ground to push bar	1000 mm
Distance from seat to the upper edge of the back	450 mm
Distance from seat to footrests	400/420/440/460/480 mm
Total weight (model 61 cm/71 cm)	27 kg
Swing away arm rests	Yes
Footrests swing away, height adjustable	Yes
Footrest material	PU foam
Individual wheel locks	Yes, 2 pieces
Turning radius	1580 mm
Frame	Powder lacquered chromium molybdenum steel
Wheels	Ø125 mm, massive, 4 pieces
Seat/back/arm rests	Polyurethane
IP classification	IPX4
ISO classification / Risk assessment	09 12 03 21 / EN/ISO 14971:2012
General requirements and test methods	EN 12182
The surface temperature can rise when the Shower Commode is exposed to external heat sources e.g., sunlight.	

Ordering

Model	Item number
Shower Commode incl. standard armrests, standard footrests, and commode bucket, 610 mm	0150-061-000
Shower Commode incl. standard armrests, standard footrests, and commode bucket, 710 mm	0150-071-000
Shower Commode incl. butterfly armrests, standard footrests, and commode bucket, 610 mm	0151-061-000
Shower Commode incl. butterfly armrests, standard footrests, and commode bucket 710 mm	0151-071-000
Butterfly arm rest right (adds 7 cm width between the arm rest)	0150-999-012R
Butterfly arm rest left (adds 7 cm width between the arm rest)	0150-999-012L
Commode bucket	0140-999-001
Full size seat for shower commode with bucket and cover, 610 mm	0150-999-022
Full size seat for shower commode with bucket and cover, 710 mm	0150-999-023
Footrest standard shower commode right	0156-999-090H
Footrest standard shower commode left	0156-999-090V
Calf support/amputation support, right	0156-999-030
Calf support/amputation support, left	0156-999-031